

TELESCOPIC CHANNEL RAMP

INSTRUCTIONS FOR USE

Metro[®]
Mobility Products

Product codes: MTR12 (4ft)
MTR15 (5ft)
MTR18 (6ft)
MTR21 (7ft)

Capacity : 300kg (660lbs)

INTENDED PURPOSE: These ramps are intended to be used as a wheelchair accessory for persons with disabilities to navigate areas of different heights.

BEFORE USE


- Remove all packaging and ensure the ramps have not been damaged in transit. Retain packaging.
- Carefully read the instructions and warning texts before using the ramps.
- Keep instructions for reference.
- On each ramp you will find a label that shows the maximum permitted load for the ramp.
- Take care when handling ramps at all times as edges can be sharp, especially where the ramp has been in contact with hard surfaces.

TO DEPLOY

1. Ensure the ground/surface at both upper and lower levels is firm and stable.
2. Release the latch as shown and fully extend both telescopic ramps. (Pic 1)
3. Check ramp is long enough to cover the step/rise.
4. Note: top and bottom ends of the ramps differ. (Pic 2 & 3)
5. Position the ramps the required width apart. The wheelchair should travel up the ramps easily.
6. Ensure the ramps do not become dislodged in use.
7. Check the ramps are firmly located and stable each time they are used.
8. To store after use, release the latch as shown and fully slide the sections back together ensuring the latch engages.



Pic 1

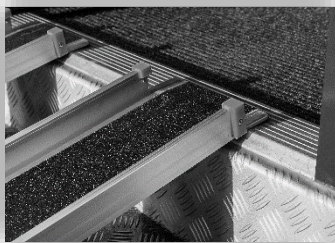

Pic 2



Pic 3

MAINTENANCE

- The ramps should be stored in a clean, dry place.
- If the ramps become damaged or slippery in any way they should not be used.
- The ramps may be cleaned with normal cleaning products and left to dry before use.
- Maximum recommended cleaning temperature 50°C; please enquire for further details on use with industrial cleaning machines.
- For disposal of this product, please liaise with a metals recycling contractor.

WARRANTY

- The ramps carry a 12-month warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- The ramps are not designed for permanent outside use and must not be fixed down.
- Any modification/repair other than by manufacturer will void the warranty.



ESL Services - Suite 27, Geddes House - Kirkton North Road - Livingston - EH54 6GU
Tel: 01506 412791 Website: <https://esl-services.co.uk>
For technical support or details on your nearest stockist please call 01506 412791